patient instructions for use
mode d'emploi pour le patient
istruzioni d'uso per il paziente
gebrauchsanweisung für den Patienten
instrucciones de uso para el paciente
instruções de uso para o paciente
οδηγίες χρήσης για τον ασθενή
brugsanvisning for patienter
gebruiksaanwijzing voor de patiënt
patientens bruksanvisning
käyttöohjeet potilaille
instrukcja stosowania dla pacjenta
návod k použití pro pacienta
használati utasítás a beteg számára
инструкции за употреба за пациента
患者使用说明
инструкции по применению для
пациента
hasta kullanım talimatları
คำแนะนำการใช้ผลิตภัณฑ์สำหรับผู้เข้ารับการฟอกสีฟัน
환자 사용 지침
إرشادات الاستخدام للمريض
患者用取扱説明書

Zoom[®]!

1	English
4	Français
7	Italiano
10	Deutsch
13	Español
16	Português
19	Ελληνικά
22	Dansk
25	Nederlands
28	Svenska
31	Suomi
34	Polski
37	Česky
40	Magyar
43	Български
46	简体中文
49	Русский
52	Türkçe
55	ภาษาไทย
58	한국어
61	العربية
64	日本語

symbols used in labeling

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Instructions for Zoom® Patient Post Care & Maintenance

Contents

- 1 Tray Case for Custom Fit Trays (made by dentist)
- 1 Relief® ACP Oral Care Gel, 2.4mL (0.08 fl. oz.), tip
- 1 NiteWhite® ACP 22% CP, or DayWhite® ACP 9.5% HP or NiteWhite® ACP Turbo 6% HP 2.4mL(0.08 fl. oz.), tip

Instructions for Post Care – Relief ACP

1. DEVICE DESCRIPTION

Active ingredients are potassium nitrate and sodium fluoride.

2. INDICATIONS FOR USE

Relief of tooth sensitivity caused by thermal or chemical stimuli, periodontal conditions and/or gum recession. For use only by adults and children 13 years of age or older.

3. APPLICATION INSTRUCTIONS

- Brush teeth and rinse
- Place small amount into each tooth compartment of tray
- Normal wear time 10-30 minutes

4. WARNINGS

- Keep out of reach of children
- See dentist if tooth sensitivity persists or worsens
- Do not use for longer than 4 weeks unless recommended by dentist

Instructions for Whitening Maintenance

1. DEVICE DESCRIPTION

The active whitening ingredients in NiteWhite ACP and DayWhite ACP take-home whitening gels are hydrogen peroxide (HP) and/or carbamide peroxide (CP). They are available in a variety of formulas:

- NiteWhite ACP: 22% CP
- NiteWhite ACP Turbo: 6% HP
- DayWhite ACP: 9.5% HP

Other active ingredients include: Potassium Nitrate, Fluoride and Amorphous Calcium Phosphate (ACP) for decreasing sensitivity, adding luster, reducing fadeback and rebuilding enamel.

2. INDICATIONS FOR USE

Indicated for use in lightening the color of natural teeth. Porcelain crowns, veneers, fillings and dentures do not lighten upon whitening.

3. CONTRAINDICATIONS

Discus Dental recommends the following whitening candidates consult medical doctor or physician for advice before use:

- Pregnant and lactating mothers
- Those currently treated by physician for serious illness or disorder e.g. immune compromised, AIDS, etc.
- Children under the age of 13

4. WARNINGS

- Remove cap safely, contents may be under pressure
- Risk of serious damage to the eyes
- Harmful when swallowed

5. FIRST AID INSTRUCTIONS

Contact a Poison Information Center or medical doctor at once if large quantity (over 25% of syringe) is swallowed. Provide ingredient label if possible. Do not induce vomiting, immediately give glass of water. If Hydrogen Peroxide (HP) or Carbamide Peroxide (CP) gel gets into eyes, on skin or hair, flush with running water.

6. PRECAUTIONS

- Avoid exposure of hypersensitive teeth to whitening gel
- Keep out of reach of children

7. TREATMENT OPTIONS (not included in all kits)

Product	Recommended Wear Time
NiteWhite ACP 22% (CP), Turbo 6% (HP)	Overnight Most patients will achieve optimal results within 1–2 weeks.
DayWhite ACP 9.5% (HP)	Day-Wear 30 minutes, twice a day. Most patients will achieve optimal results within 2 weeks.

8. APPLICATION INSTRUCTIONS

- Floss and brush teeth
- Twist and pull off clear plastic cap in counterclockwise motion
- Place mixing nozzle on syringe and secure by twisting in clockwise motion
- Place small drop of gel in each tooth compartment of tray

Important: Overloading trays with gel may cause temporary gum irritation, a little gel goes a long way.

- Place tray with gel in mouth. “Bubbling” within tray is normal.
- Remove excess gel with cotton swab or dry toothbrush
- Remove mixing nozzle from syringe and replace with cap for storage after use to ensure ingredients in each side of barrel stay separate
- Rinse trays with cold water. If necessary, use toothbrush to remove residual gel. Place trays in storage case in cool dry place.
- Rinse and brush teeth to remove excess gel

Notes: Do not eat, drink or smoke while wearing trays. Area of tooth closest to gums may take longer to lighten than biting edge. It is normal to see dark colors in trays where you have amalgam (silver) fillings. Gel oxidizes surface stains on amalgam fillings.

9. POTENTIAL ADVERSE EFFECTS

Adverse effects that may be associated with use of a whitening product containing peroxide include, but are not limited to:

Sensitivity

- Patients more susceptible to sensitivity are those with exposed root surfaces and dentin, defective restorations, oral tissue injury and/or periodontal disease
- Some patients may experience increased tooth sensitivity to cold during treatment or have non-specific sensitivity in teeth, gums, tongue, lips, or throat. Symptoms should subside within 1-3 days after interrupting treatment.

10. STORAGE INSTRUCTIONS

Store between 10°C –20°C (50°F – 68°F) in a dark, dry place. Storage temperature above 20°C (68°F) will result in reduced shelf life.

11. DISPOSAL INSTRUCTIONS

Dispose of device in accordance with local waste regulations.

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Discus Dental, LLC