

Tapered Implants Subcrestal Surgical Protocol Quick Reference Guide



Tapered Implants Subcrestal Surgical Protocol


Quick Reference Guide

NOTE:


- The recommended drill speed for all drills is 1200 – 1500rpm.
- The Quad Shaping Drills must be used without pumping actions.
- The recommended implant placement speed is 15 – 20rpm.
- The implant placement torque may exceed 50Ncm.
- Tapping is required for implant placement in dense bone (Type I).
- Undersizing the osteotomy by one shaping drill diameter is recommended for implant placement in soft bone (Type IV).
- Hand ratcheting may be necessary to fully seat the implant in the osteotomy.
- Certain® Internal Connection Driver Tips should be inspected for wear before use.
- It is recommended that reusable drills be replaced after 15 uses.

D = Diameter
C = Collar
P = Platform
L = Length

ACT® Twist Drill Depth Marks

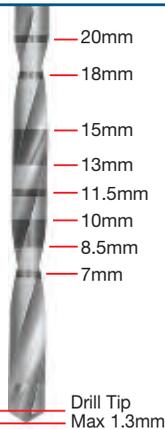


Tapered Certain Internal Connection & External Hex Connection 3.25mm Diameter Implants

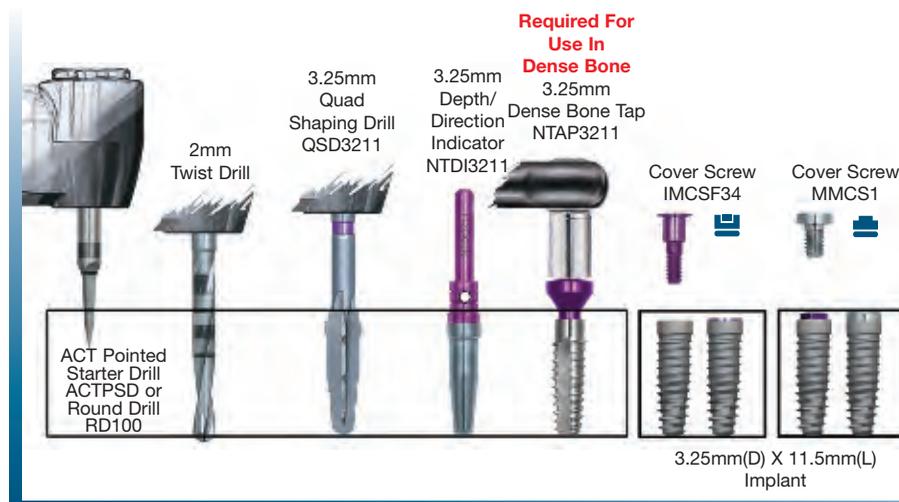

IMPORTANT NOTE!


Exceeding insertion torque of more than 90Ncm may deform or strip the driver tip or the implant's internal hex and may possibly delay the surgical procedure.

Tapered Certain PREVAIL® 4/3mm, Tapered Certain Internal Connection & External Hex Connection 4mm Diameter Implants

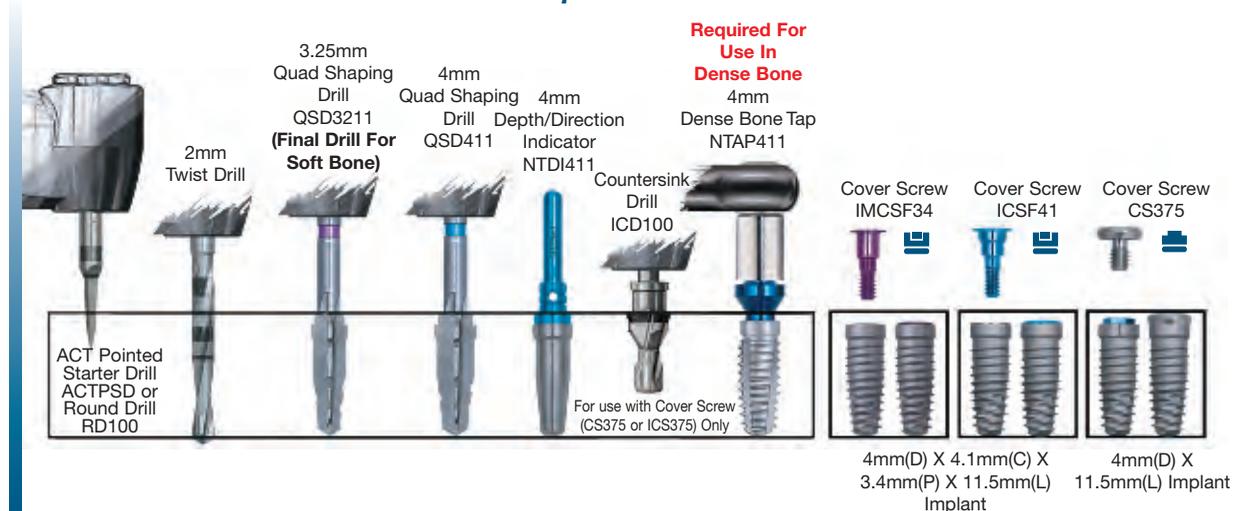


D = Diameter
C = Collar
P = Platform
L = Length



Tapered Certain® PREVAIL® 5/4mm, Tapered Certain Internal Connection & External Hex Connection 5mm Diameter Implants

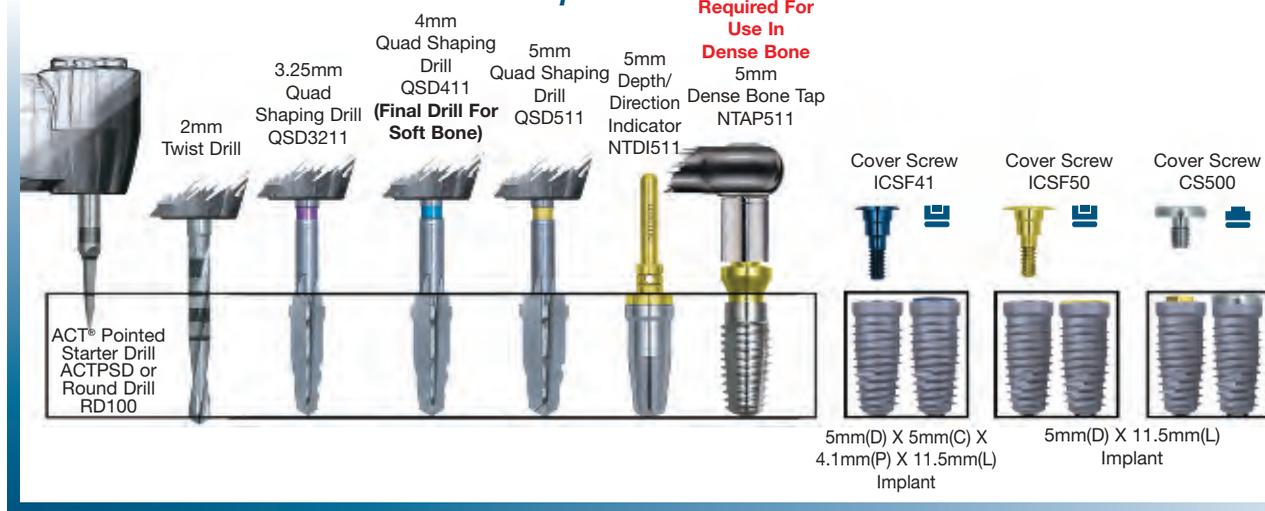

Tapered Certain PREVAIL 6/5mm, Tapered Certain Internal Connection & External Hex Connection 6mm Diameter Implants

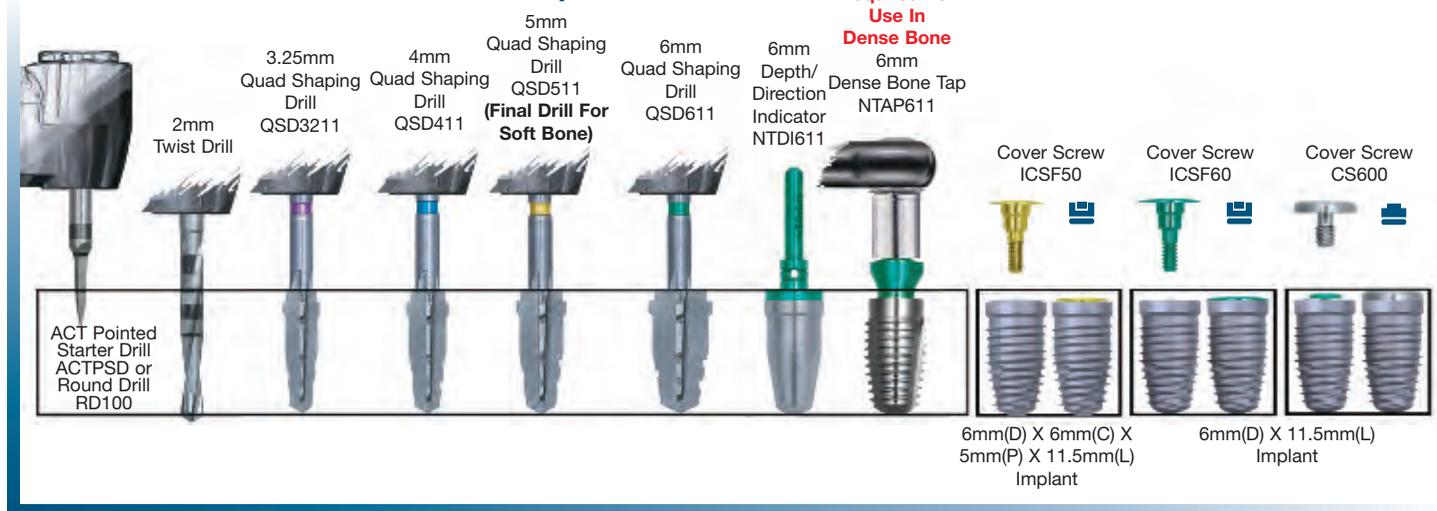

Additional Placement Protocol Considerations

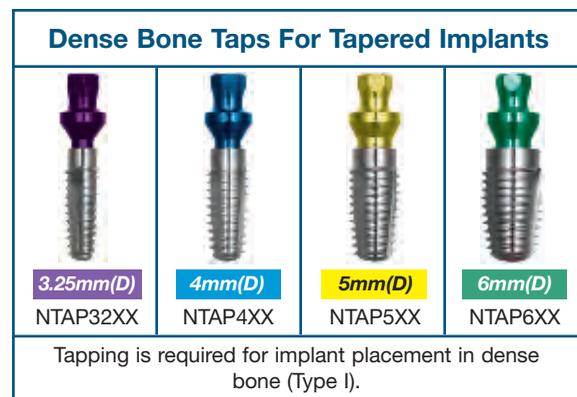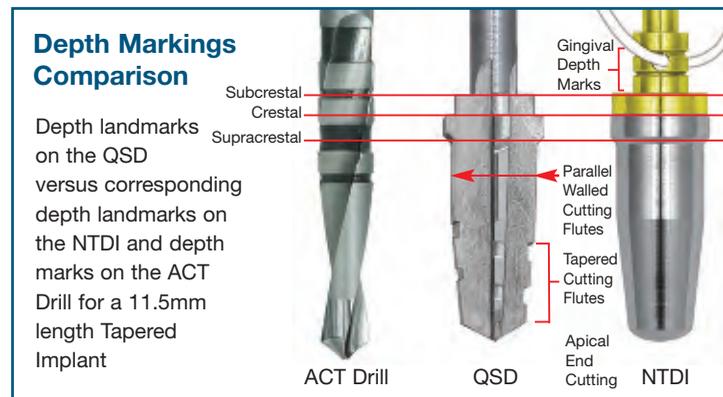

**WANT MORE TREATMENT
OPTIONS?**

**PLAN FOR IMMEDIATE
PRIMARY STABILITY**

www.biomet3i.com

Not Available In All Markets. Please Contact Your Local BIOMET 3i Sales Representative For Availability Or Visit www.biomet3i.com.

BIOMET 3i™



Global Headquarters
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Outside the U.S.: +1-561-776-6700
Fax: +1-561-776-1272
www.biomet3i.com

EC REP
BIOMET 3i
Europe, Middle East & Africa
WTC Almeda Park, Ed. 1, Planta 1ª
Pl. de la Pau, s/n
08940, Cornellà de Llobregat
(Barcelona) Spain
Phone: +34-93-470-55-00
Fax: +34-93-371-78-49



ART972